Aquatec Step

en	Stepping aid User Manual	1
de	Einstiegshilfe Gebrauchsanweisung	3
fr	Marchepied Manuel d'utilisation	4
it	Ausilio per salita Manuale d'uso	5
es	Escalón de ayuda Manual del usuario	7
pt	Dispositivo de escalada Manual de utilização	8
nl	Opstapje Gebruiksaanwijzing	9
sv	Pall Bruksanvisning	11
no	Badekrakk Bruksanvisning	12
da	Skammel Brugsanvisning	13
fi	Korokeapuväline Käyttöohje	14

en

I General

I.1 Images

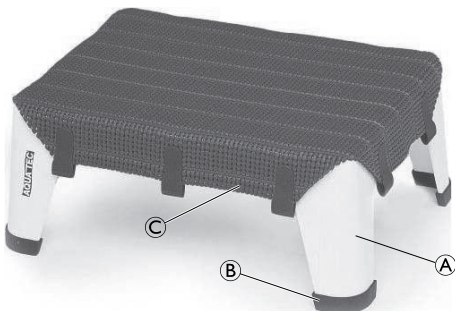
Fig. 1

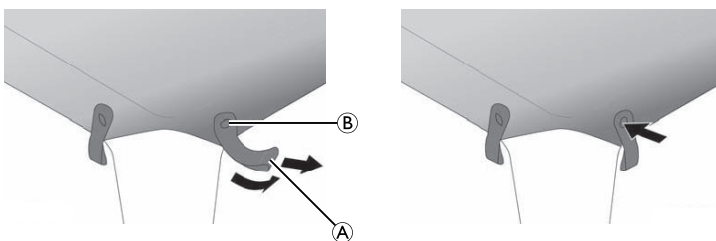
Fig. 3

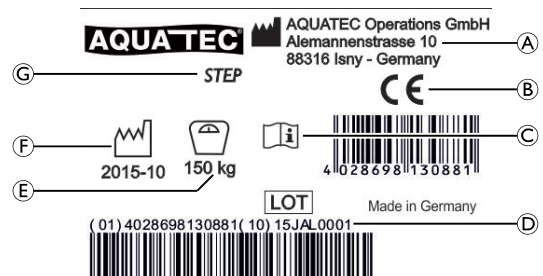
Fig. 2

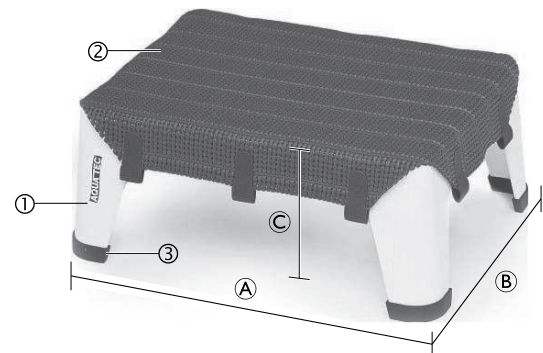
Fig. 4

This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

AQUATEC

Yes, you can.®

1.2 Symbols in this user manual

In this user manual, warnings are marked with symbols. The title next to the warning symbol indicates the level of danger.

WARNING

Indicates a dangerous situation which could lead to death or serious injury if not avoided.

IMPORTANT

Indicates a dangerous situation which could lead to damage if not avoided.

Useful tips, recommendations and information to ensure efficient and smooth operation.

This product complies with the directive 93/42/EEC for medical products. The launch date for this product is specified in the CE declaration of conformity.

Manufacturer

1.3 Intended use

The stepping aid is to be used solely as an aid in the bathroom, e.g. for getting into the bath tub. Any other use is prohibited.

Operating temperature: 10 – 40 °C

1.4 Scope of delivery (Fig. 1)

The Aquatec® Step is available with or without the non-slip, removable cover option.

The following components are included in the scope of delivery:

Ⓐ	Frame
Ⓑ	Non-slip, removable rubber caps
Ⓒ	Non-slip, removable cover (optional)
	User manual

Ⓐ, Ⓑ and optionally Ⓒ are pre-assembled.

1.5 Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the appliance was obtained.

1.6 Standards and regulations

Quality is of the utmost importance for our company; all processes are based on the ISO 9001 and ISO 13485 standards.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class I.

The product meets the requirements of the standards DIN EN 12182 (Technical aids for disabled persons - general requirements and test methods), DIN EN ISO 14971 (Medical devices - Application of risk management to medical devices) and DIN EN ISO 10993-1 (Biological evaluation of medical devices - Part 1: Evaluation and testing within a risk management system).

Invacare® is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum. We only use REACH compliant materials and components.

For further information please contact Invacare® in your country (for addresses, see back page of this manual).

1.7 Service life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

2 Safety

2.1 Safety information

WARNING! Risk of falling!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Do not move or displace the product under load or during use.

2.2 Labels and symbols on the product (Fig. 2)

Ⓐ	Manufacturer	Ⓔ	Load capacity
Ⓑ	Conformity mark	Ⓕ	Production date
Ⓒ	Note (observe accompanying documents)	Ⓖ	Product designation
Ⓓ	Lot number		

The identification label is attached to the underside of the product.

3 Removing and fitting the non-slip cover (Fig. 3)

1. To remove the cover, pull the loops Ⓐ until the snap fasteners Ⓑ come off.
2. To fit the cover, close all snap fasteners.

WARNING! Risk of slipping

- Make sure that all 10 snap fasteners are securely fitted before using the product.

4 Maintenance

4.1 Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined in this user manual are adhered to.

WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

4.2 Cleaning and disinfection

IMPORTANT!

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Health Care Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CG139 and your local infection control policy.
- Clean by hand.
 - Do not use any aggressive (pH value under 4.5 or over 8.5) or abrasive cleaning agents.
 - Max. cleaning temperature 60 °C.

The frame and the non-slip rubber caps can be cleaned and disinfected using commercially available agents.

1. Remove the non-slip cover (if included) and machine-wash at max. 60 °C.
2. Remove the non-slip rubber caps.

- Clean and disinfect the frame and the non-slip rubber caps by wiping down all generally accessible surfaces.
- Re-attach the rubber caps and the cover.

4.3 Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly (® 4.2 Cleaning and disinfection, page 2).

4.4 Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

5 Technical data (Fig. 4)

Dimensions and weight		Materials *	
Ⓐ	410 mm	①	PVC
Ⓑ	310 mm	②	PP
Ⓒ	165 mm	③	TPE
Product weight	1.5 kg		
Max. user weight	150 kg		

* All product components are latex free.

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Invacare A/S
Aquatec Operations GmbH