


PACKING INSTRUCTIONS

Küschall Champion SK Travel Bag

This travel bag is only suitable for the Küschall Champion SK wheelchair with the folding front frame.

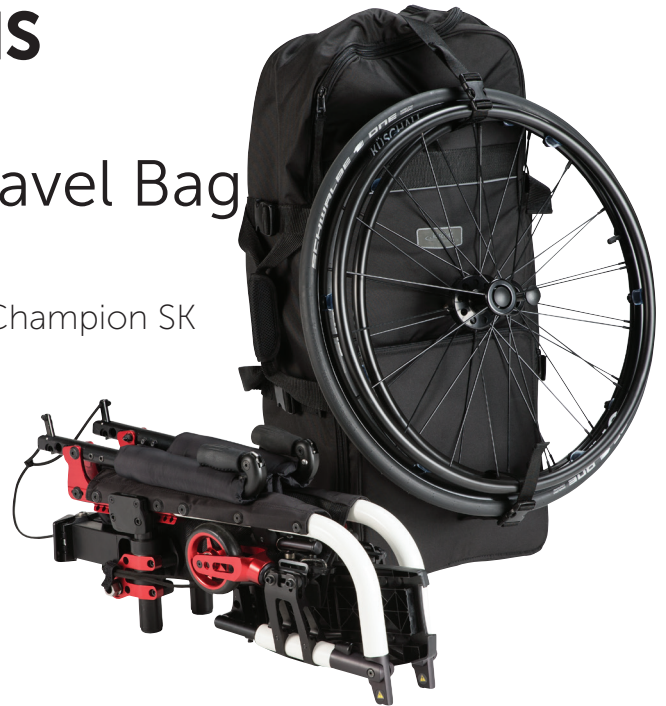

Handling



The two rear wheels can be attached with straps to the closed travel bag. The camber axles should be stored within the bag.



Three types of handles allow the bag to be carried in any position needed:

- Backpack
- Top holder
- Hold all



For comfortable posture and easy transportation, all straps attached to the bag should be fastened as tightly as possible.



Fasten the cushioned shoulder straps so they fit as tightly to your body as possible to prevent back pain.

Open the bag completely and open all straps for simple packing. Ensure that the straps inside the bag are also open.



Top (1)



Straps (3)

Bottom (2)

Put the completely folded wheelchair into the bag with the backrest lying on the bottom. The front wheels must face upwards.

The rear part of the wheelchair with the hinge part of the folding mechanism must be placed at the bottom part of the bag (2).

Use the central internal straps of the bag to fasten the wheelchair tightly to the bottom of the bag (3).



If the wheelchair has fixed clothes guards, ensure that the bags separating tabs are placed between the carbon parts and the axle holes to protect the surfaces.



If the wheelchair has removable clothes-guards or mudguards, use the external front pocket with separating tabs to store the side parts to protect the surfaces.