
Aquatec® Trans

en	Transfer board with turnable disk User Manual	1
de	Brett zum Umsetzen mit Drehscheibe Gebrauchsanweisung	3
fr	Planche de transfert avec disque tournant. Manuel d'utilisation	4
it	Asse di trasferimento con disco girevole Manuale d'uso	6
es	Placa de traslado con disco giratorio Manual del usuario	7
pt	Placa de transferência com disco giratório Manual de utilização	8
nl	Overstaplaaat met draaibare schijf Gebruiksaanwijzing	9
da	Flytteplade med drejeskive Brugsanvisning	10
no	Forflytningsbrett med dreibar plate Bruksanvisning	11
sv	Överflyttningsplattform med vridplatta Bruksanvisning	12
fi	Siirtolauta, jossa käännettävä levy Käyttöohje	14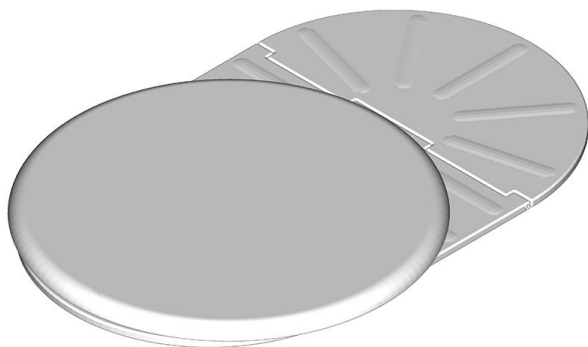

This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

AQUATEC®

Yes, you can.®

I General

I.1 Images

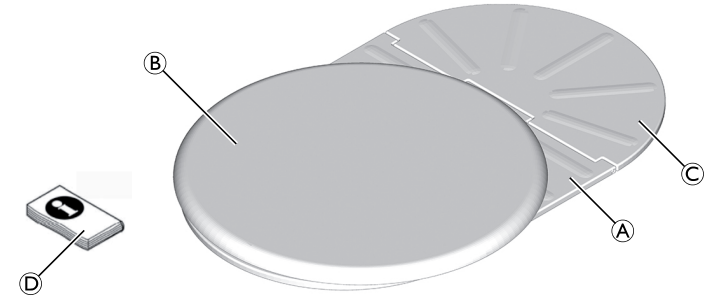
Fig. 1

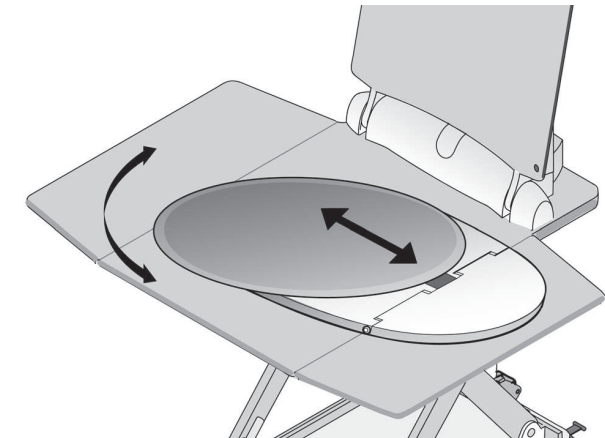
Fig. 3

I.2 Intended use

The transfer board with rotary seat is intended to be used in the bathroom for people with sufficient torso stability in combination with a bath lift. The product provides a movable and turnable seating surface and serves as a transfer aid from the bath tub rim to the bath lift and back. Any other use is prohibited.

Operating temperature: 10 – 40 °C

I.3 Scope of delivery (Fig. 1)

The following components are included in the scope of delivery:

A	Main board
B	Turnable disk (pre-mounted)
C	Side flap (pre-mounted)
D	User manual

I.4 Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the appliance was obtained.

I.5 Standards and regulations

Quality is of the utmost importance for our company; all processes are based on the ISO 9001 and ISO 13485 standards.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class I.

The product meets the requirements of the standards DIN EN 12182 (Technical aids for disabled persons - general requirements and test methods), DIN EN ISO 14971 (Medical devices - Application of risk management to medical devices) and DIN EN ISO 10993-1 (Biological evaluation of medical devices - Part 1: Evaluation and testing within a risk management system).

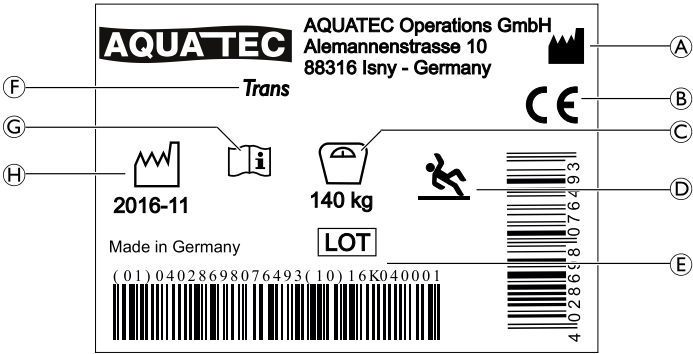
Fig. 2

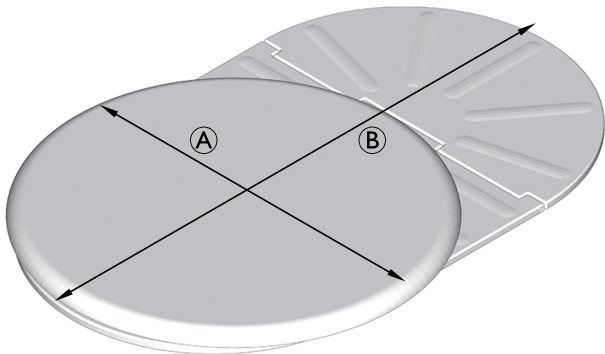
Fig. 4

Invacare® is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum. We only use REACH compliant materials and components.

For further information please contact Invacare® in your country (for addresses, see back page of this manual).

I.6 Service life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary depending on frequency and intensity of use.

2 Safety

2.1 Safety information

WARNING!
Risk of falling!

- Do not stand on this product.
- Do not use as a climbing aid.

WARNING!
Risk of injury!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Do not move or displace the product under load or during use.

2.2 Labels and symbols on the product (Fig. 2)

A	Manufacturer address
B	Conformity mark
C	Load capacity

Ⓓ	Symbol - Risk of falling
Ⓔ	Lot number
Ⓕ	Product designation
Ⓖ	Read user manual
Ⓗ	Date of manufacture

The identification label is attached to the underside of the product.

3 Setup and usage (Fig. 3)

1. Place the transfer board with turnable disk on the seating surface of the bath lift, so that the side flaps lie on top of each other.

The turnable disk is unlocked by the weight of the user.

2. Slide the turnable disk onto the side flap towards the bath tub rim.
3. Sit down on the turnable disk and carefully slide fully inwards on the bath lift seat.
4. Rotate by 90 degrees into the bath tub so that you are correctly positioned for using the bath lift.

4 Maintenance

4.1 Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined in this user manual are adhered to.

WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

4.2 Cleaning and disinfection

IMPORTANT!

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Health Care Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CG139 and your local infection control policy.

- Clean by hand.
- Do not use any aggressive (pH value under 4.5 or over 8.5) or abrasive cleaning agents.
- Max. cleaning temperature 60 °C.

The product can be cleaned and disinfected using commercially available agents.

1. Disinfect the product by wiping down all generally accessible surfaces with disinfectant.

4.3 Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly (® 4.2 Cleaning and disinfection, page 3) and check the safety functions.

4.4 Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

5 Technical data (Fig. 4)

Ⓐ	365 mm
Ⓑ	516 mm
Height	25 mm
Product weight	2.2 kg
Max. user weight	140 kg
Materials*	PUR / ABS

*All product components are corrosion resistant and latex free.

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Invacare AG
Aquatec Operations GmbH