Aquatec® product range

9630E Styxo, Aquatec 90 / Aquatec 900 / Aquatec 90000,
Pico / Pico TS, Tellus / Pluto, Sorrento range

en	Hygiene medical devices Service Manual	3
de	Medizinische Hygieneprodukte Servicehandbuch	15
fr	Dispositifs médicaux d'hygiène Manuel de maintenance	27
da	Medicinsk hygiejneudstyr Servicemanual	39
nn	Medisinsk hygieneutstyr Servicehåndbok	51
sv	Hygientekniska medicinska produkter Servicemanual	63
fi	Lääkinnälliset hygienialaitteet Huoltokäsikirja	75

DEALER: Keep this manual.
The procedures in this manual MUST be performed by a qualified technician.

AQUATEC®

Yes, you can.®

©2018 Invacare Corporation

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

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Contents

- 1 General 4**
 - 1.1 Introduction 4
 - 1.2 Symbols in this manual 4
- 2 Safety 5**
 - 2.1 General safety information 5
 - 2.2 Personal safety information 5
- 3 Overview 6**
- 4 Servicing 10**
 - 4.1 Inspection checklist 10
 - 4.2 Spare parts 10
 - 4.3 Colour codes 10
- 5 Reconditioning 11**
 - 5.1 Cleaning and disinfection 11
 - 5.2 Material overview 11
 - 5.3 Reconditioning scheme 11
- 6 General repair information 14**

1 General

1.1 Introduction

This document contains important information about assembly, adjustment and advanced maintenance of the product. To ensure safety when handling the product, read this document and the user manual carefully and follow the safety instructions.

Find the user manual on Invacare's website or contact your Invacare representative. See addresses at the end of this document.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

For pre-sale and user information, see the user manual.

For more information about the product, for example product safety notices and product recalls, contact your Invacare representative. See addresses at the end of this document.

1.2 Symbols in this manual

Symbols and signal words are used in this manual and apply to hazards or unsafe practices which could result in

personal injury or property damage. See the information below for definitions of the signal words.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

Manufacturer of the product.

2 Safety

2.1 General safety information

WARNING!

Risk of injury or damage to property

- The procedures in this manual must only be performed by a qualified technician.
- Use only original accessories and spare parts.
- Do not handle this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals, installation manuals or instruction sheets supplied with this product or optional equipment.
- After each assembly, check that all fittings are properly tightened and that all parts have the correct function.

WARNING!

Risk of contamination

- Clean and disinfect the product before servicing.

IMPORTANT!

- Some replacement parts are only available as a kit. Always use the complete new kit when replacing a part.
- Spare parts can be ordered from Invacare. Refer to your local Invacare website to access the electronic spare parts catalogue (ESPC).

IMPORTANT!

- Refer to the user manual of this product for information on
- Technical data
 - Product components
 - Labels
 - Additional safety instructions
 - Cleaning and disinfection instructions

The information contained in this document is subject to change without notice.

2.2 Personal safety information

These safety instructions are intended to help avoid accidents during work and must be observed under all circumstances.

All employees coming into contact with contaminated products must regularly consult a company doctor. Work clothing and personal protective equipment must be available in necessary quantities and be in proper condition. Reliable hand and surface disinfection must be ensured.

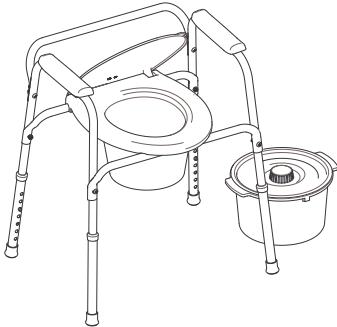
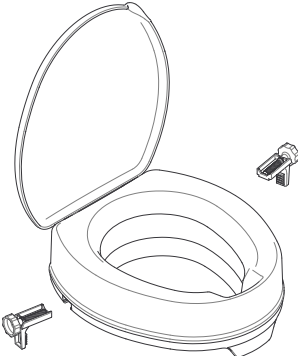
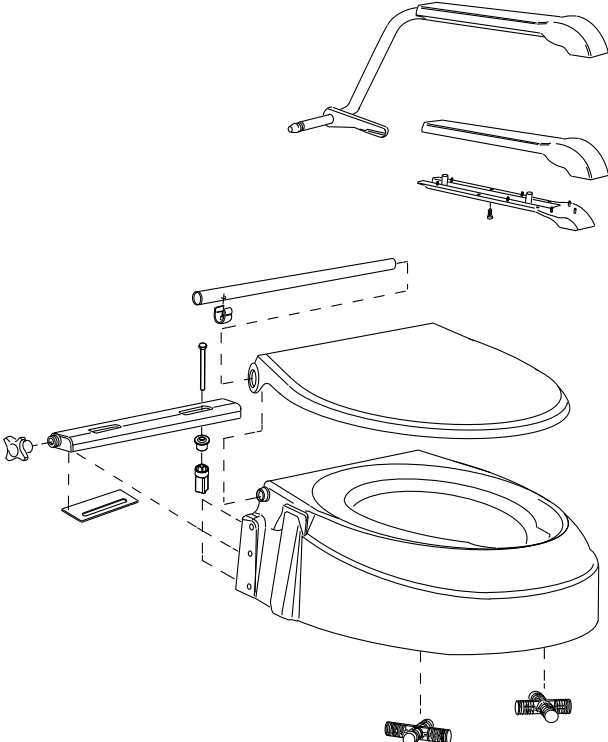
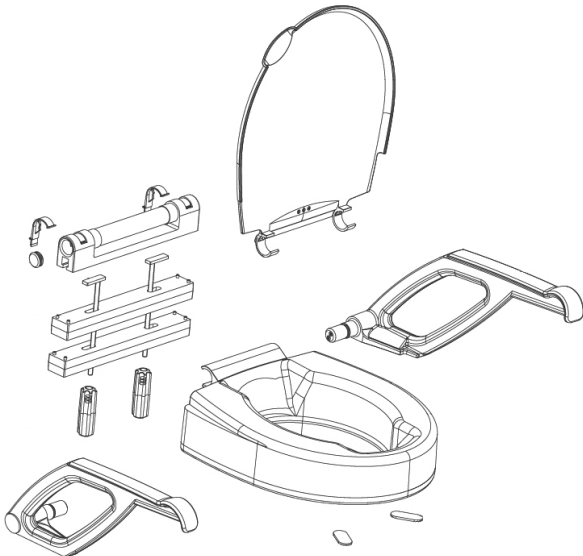
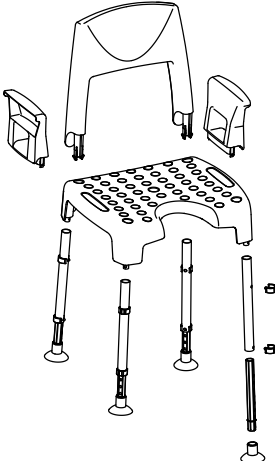
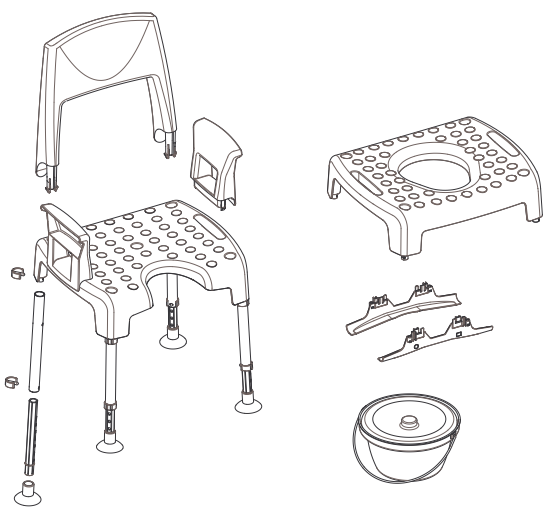
WARNING!

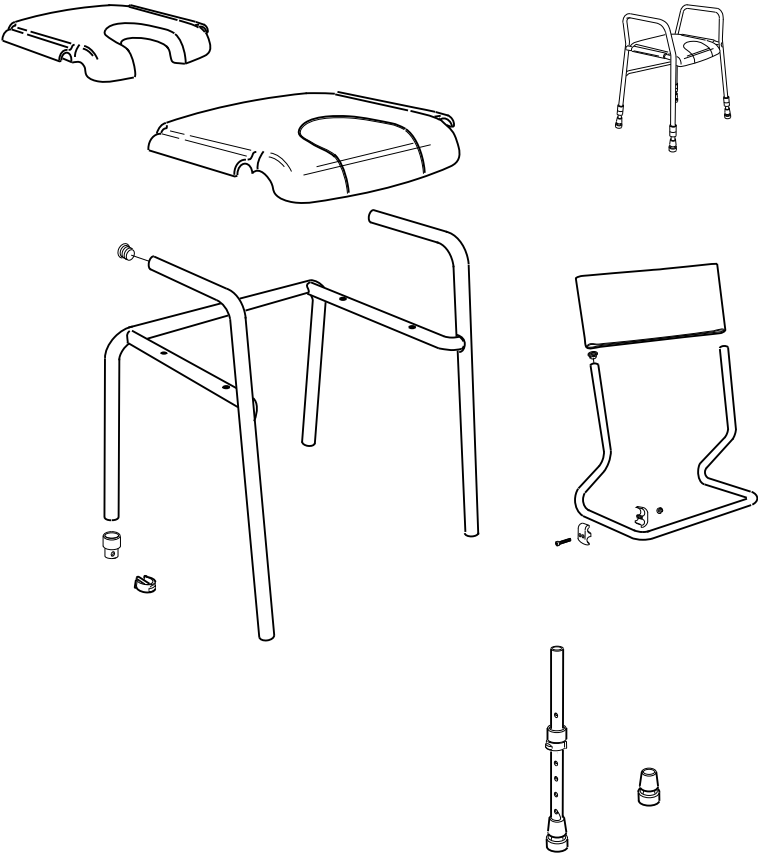
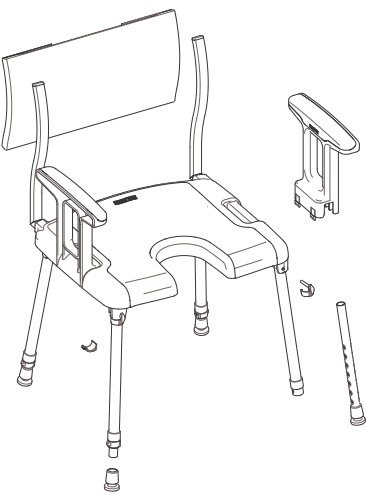
Risk of contamination

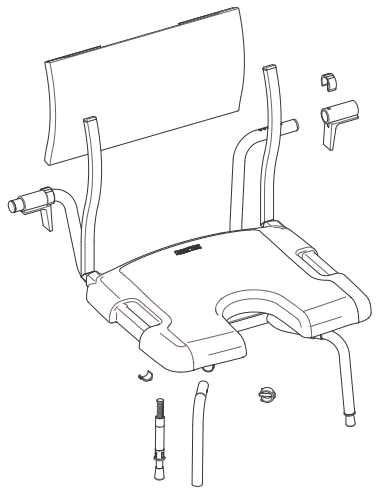
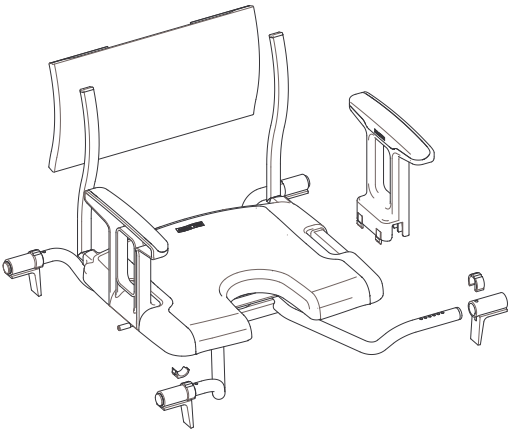
- Clean and disinfect the product before carrying out repairs.

3 Overview

Product name	Product view
9630E Styxo	
Aquatec 90	
Aquatec 900	

Product name	Product view
Aquatec 90000	 This diagram shows the components of the Aquatec 90000 shower seat. It includes a large, curved backrest, a rectangular seat base with a central opening, two side armrests, and a mounting bracket. Several screws and small plastic spacers are also shown, indicating how the parts are assembled.
Pico	 This diagram illustrates the Pico shower stool. It features a perforated plastic seat, a curved backrest, and two side armrests. The seat is supported by four adjustable legs, each with a footrest. The diagram shows the seat, backrest, armrests, and legs with their respective mounting hardware.
Pico TS	 This diagram shows the Pico TS shower stool. It includes a perforated seat, a backrest, and armrests. The seat is mounted on a base with four legs. The diagram also shows a separate circular component, likely a footrest or a base plate, and the hardware needed for assembly.

Product name	Product view
Pluto / Tellus	
Sorrento shower chair	

Product name	Product view
Sorrento bathtub seat	
Sorrento bathtub swiveling seat	

4 Servicing

4.1 Inspection checklist

General inspection	☺	☹
Has the product been cleaned and disinfected?	☐	☐
Is the product in good condition and is it complete (product and accessories)?	☐	☐
Is the product free from damage or weaknesses of any kind?	☐	☐
Does the product operate correctly under nominal load?	☐	☐
Is the product fully functional in accordance with the user manual?	☐	☐
Eradication of faults	☺	☹
Have all the faults found been eliminated and have faulty components been replaced?	☐	☐
Completion of checks	☺	☹
Has the product been cleaned and disinfected?	☐	☐
Are all screws/bolts firmly fitted and is the product securely assembled	☐	☐
Is the product technically and functionally safe?	☐	☐

Is the identification label easily readable and is it securely mounted on the product?	☐	☐
Is the product accompanied by the latest revision of the user manual?	☐	☐

4.2 Spare parts

WARNING!

Original spare parts must be used in all repairs. Otherwise the warranty and product declaration of conformity shall be rendered invalid.

All spare parts must be obtained from the Invacare customer service department. An electronic spare parts catalogue can be found at www.invacare.co.uk.

4.3 Colour codes

Item	Colour code (RAL)
Styxo – frame	Cool grey, Pantone 7C
Aquatec 900 – armrest tubes	White, RAL9003
Tellus/Pluto – frame	White, RAL9003
Tellus/Pluto – Clamp for backrest	Blue, RAL 5002

5 Reconditioning

5.1 Cleaning and disinfection

IMPORTANT!

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Healthcare Environments, please refer to your local guidelines on infection control and your local infection control policy.

Preparation at the place of use

1. Thorough pre-clean the product with hot tap water.

- Pre-cleaning storage time must be as short as possible.
- Products to be cleaned must be protectively packed, e.g. sealed with adhesive tape in such a way that no more openings are visible on the packaging.
- Pathogens with which the product has been infected (even potentially) must be noted and visibly specified on the outer packaging.

Preparation before cleaning

1. Dismantle the product into individual components and create an optimum item for cleaning and disinfection.
2. If present, cover electrical connections with protective caps and seal any sites where water may leak in.
3. Remove superficial contamination.

Cleaning

1. Clean the product with decalcifying sanitizing detergent. Manual brushing is permitted.
2. Rinse with tap water until the detergent is completely removed.
3. Mechanically cleaning of washable components (e.g. covers) in industrial washing machine.

4. Rinse with tap water until the detergent is completely removed.
5. Manually clean the product with universal detergent. Supplementary manual brushing is also permitted.
6. Rinse with tap water until the detergent is completely removed.
7. Air dry until there is no more superficial moisture on the product.

WARNING!

Risk of contamination

- Do not continue to use the product in case of water penetration or if it shows discolourations which cannot be removed.

Disinfection

1. Disinfect the product by wiping generally accessible surfaces with disinfectant.
2. Air dry until there is no more superficial moisture on the product.

5.2 Material overview

Products	Material(s)
Styxo	PP, Coated steel
Aquatec 90	PP
Aquatec 900	PP, Coated steel
Aquatec 90000	PP
Pico / Pico TS	PP, Aluminum
Tellus / Pluto	PP, Coated steel
Sorrento range	PP (glass fiber reinforced)/TPE, Aluminum, Nylon (PVC coated)

5.3 Reconditioning scheme

For replacement parts refer to the electronic spare parts catalogue (ESPC).

The following items must be reviewed and checked when reconditioning is required:

Toilet frames

Products	Issue	Troubleshooting action	OK
Styxo 9630E	Frame damaged (cracks, fractures)	Discard the product.	☐
	Frame coating damaged or scratched.	Use touch-up colours to repair damages. See chapter 4.3 <i>Colour codes, page 10</i> .	☐
	Toilet seat damaged (cracks, sharp edges)	Replace with new one. See ESPC.	☐
	Toilet pan and/or lid damaged		
	Splash guard damaged		
	Armrest pad damaged	Replace with new one. See ESPC.	☐

Products	Issue	Troubleshooting action	OK
	Damage to parts of the height adjustable legs	Replace plastic cap. See ESPC. Replace locking knob. See ESPC. Replace rubber foot. See ESPC.	☐
	Screws loose	Re-tighten all screws	☐

Toilet seat raisers

Products	Issue	Troubleshooting action	OK
Aquatec 90	Lid and/or hinge damaged	Replace with new one. See ESPC.	☐
Aquatec 900 Aquatec 90000	Installation kit/ clamping elements damaged	Replace with new one. See ESPC.	☐
Aquatec 90 Aquatec 900	Body damaged	Discard the product.	☐
Aquatec 900	Crossbar/ attachment cross/ clips/ rear tube/ adhesive tapes/ star screw damaged	Replace with new one. See ESPC.	☐
	Armrests/ armrest overlays damaged		☐
Aquatec 90000	Armrests damaged	Replace with new ones. See ESPC.	☐
	Body damaged	Replace with new one. See ESPC.	☐
	Slip protection/ base/ clip/ height adapter/ plug damaged	Replace with new one. See ESPC.	☐

Shower stools / Shower chairs / Commodes

Products	Issue	Troubleshooting action	OK
Pico Pico TS	Backrest/ armrest/ seat plate damaged	Replace with new one. See ESPC.	☐
	Leg damaged	Replace complete leg or affected part with new one. See ESPC.	☐
Pico TS	Toilet pan/ holder damaged	Replace with new one. See ESPC.	☐
	Toilet pan holder loose	Re-tighten the screws.	☐
Pluto / Tellus	Frame tube / backrest tube damaged	Discard the product.	☐
	Frame end cap damaged	Replace with new one. See ESPC.	☐
	Seat plate damaged	Replace with new one. See ESPC.	☐
	Backrest cover damaged (Pluto only)	Replace with new one. See ESPC.	☐
	Backrest fixation kit loose	Re-tighten screws.	☐
	Backrest fixation kit broken	Replace complete fixation kit. See ESPC.	☐
	Telescope leg damaged	Replace complete telescope leg. See ESPC.	☐
	Transition cap and/or clip damaged	Replace with new one. See ESPC.	☐
	Rubber tip damaged	Replace with new one. See ESPC.	☐
	Frame coating damaged or scratched.	Use touch-up colours to repair damages. See chapter 4.3 <i>Colour codes, page 10.</i>	☐

Products	Issue	Troubleshooting action	OK
Sorrento shower chair Sorrento bathtub (swivel) seat	Seat damaged	Discard the product.	☐
	Armrest damaged	Replace with new one. See ESPC.	☐
	Backrest tube damaged		☐
	Backrest cover damaged		☐
	Clamps broken/damaged		☐
Sorrento shower chair	Telescope leg/ clip fixation/ clip for height adjustment/ rubber foot damaged	Replace with new one. See ESPC.	☐
Sorrento bathtub seat	Height adjustment leg damaged	Replace with new one. See ESPC.	☐
	Leg tube damaged		☐
	Clip for height adjustment damaged.		☐
Sorrento bathtub (swivel) seat	Footrest damaged	Replace with new one. See ESPC.	☐
	Clip for footrest damaged		☐

6 General repair information

IMPORTANT!

- If possible, continue to use the old identification label; if this is not possible, the new identification label must contain the same information and the old serial number. (Replacement of spare parts with serial numbers).
- When components are replaced it is necessary to ensure the traceability of the components replaced.
- If screws with thread locker are loosened, these must be replaced with new screws with thread locker. Alternatively, new thread locker must be applied (e.g. Loctite 2701).
- Parts that become damaged during removal must be replaced with new ones.

For instructions on how to replace the spare parts refer to the graphics in the ESPC.

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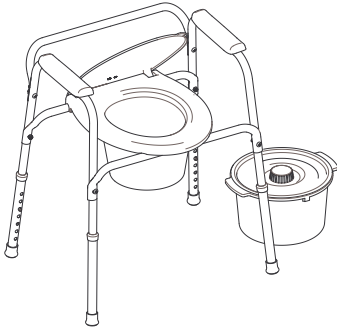
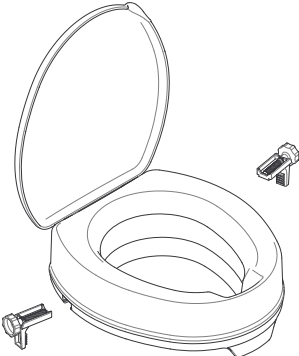
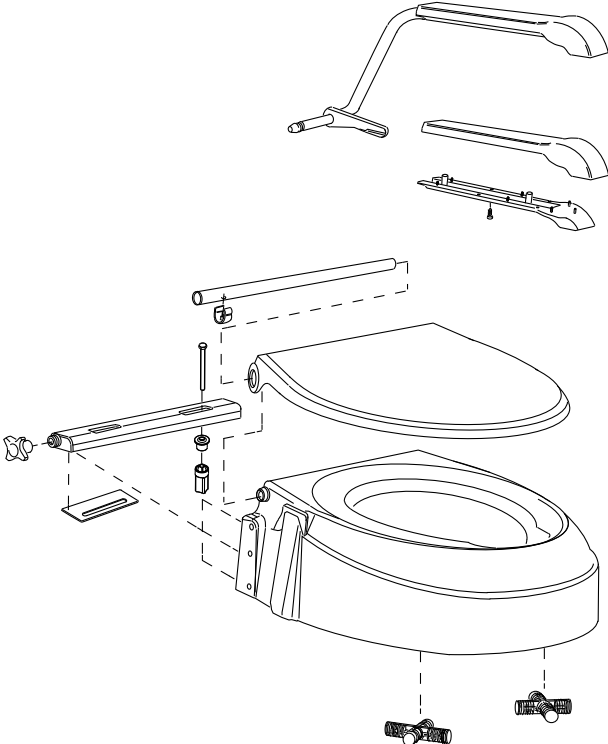
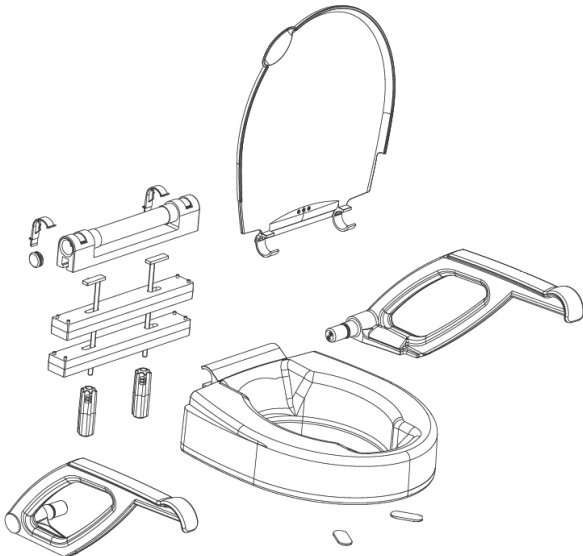
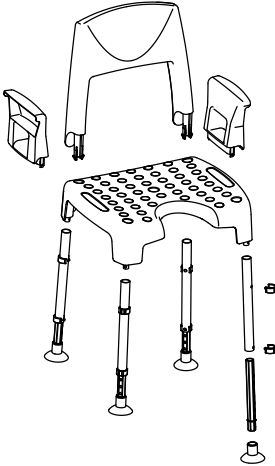
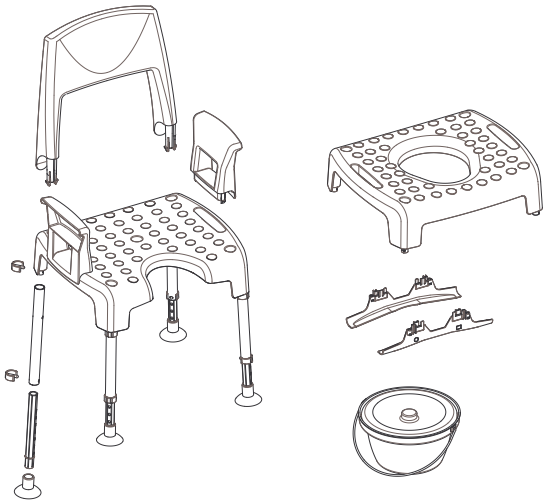
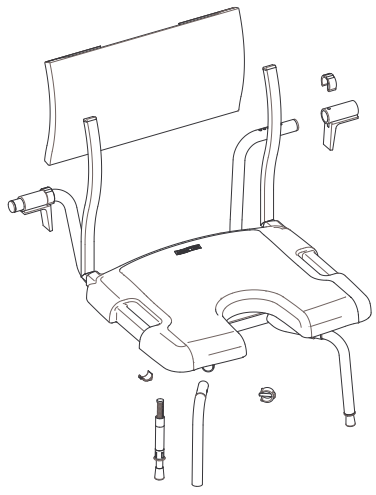
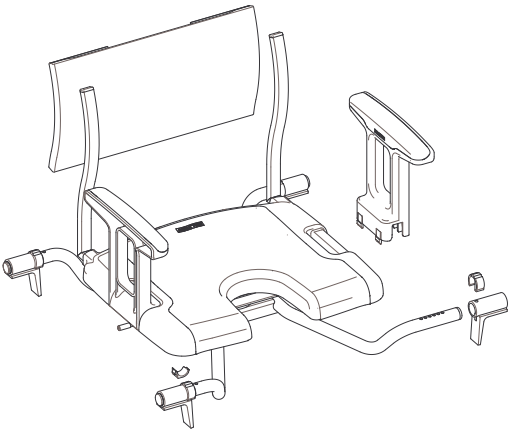
Produktnavn	Produktillustrationer
Aquatec 90000	 This diagram shows the exploded view of the Aquatec 90000 shower seat. The components include a large, curved backrest, a rectangular seat base with a central opening, two side armrests, and a mounting bracket. Several screws and small plastic spacers are also shown, indicating how the parts are assembled together.
Pico	 This diagram illustrates the exploded view of the Pico shower stool. It features a square seat with a grid of circular holes, four adjustable legs with foot caps, a backrest, and two side armrests. The diagram shows how these components are connected using screws and spacers.
Pico TS	 This diagram shows the exploded view of the Pico TS shower stool. It includes a square seat with a grid of circular holes, four adjustable legs with foot caps, a backrest, and two side armrests. Additionally, there is a separate circular component, possibly a lid or a base, and some small hardware like screws and spacers.

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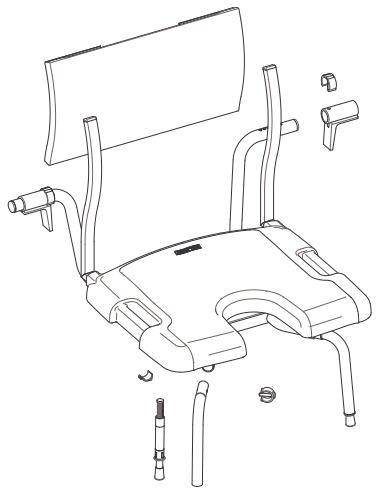
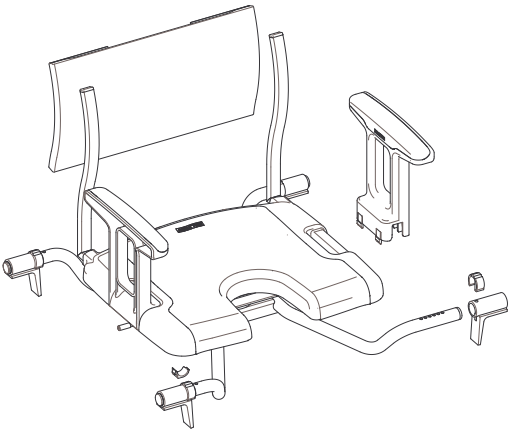
Produktnavn	Produktvisning
Sorrento badekarsete	 Exploded view diagram of the Sorrento bathtub seat. The main component is a rectangular seat with a curved backrest and integrated armrests. Surrounding the seat are various assembly parts: two long vertical support poles, two shorter horizontal poles, a small rectangular panel, and several screws and bolts.
Sorrento svingsete for badekar	 Exploded view diagram of the Sorrento swivel bathtub seat. The main component is a rectangular seat with a curved backrest and integrated armrests. Surrounding the seat are various assembly parts: two long vertical support poles, two shorter horizontal poles, a small rectangular panel, and several screws and bolts.

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Invacare New Zealand Ltd
Aquatec Operations GmbH